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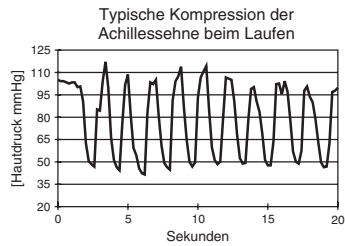
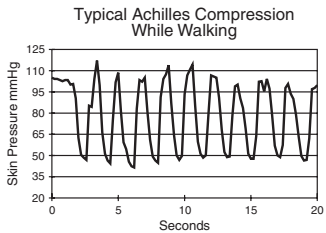
AirHeel™

The **Aircast AirHeel™** is intended for plan-
tar fasciitis and Achilles tendonitis. The
wrap applies safe, intermittent compres-
sion and support to minimize swelling and
relieve pain. This massaging compression
allows for an early return to function with
comfort and mobility. A low profile and
lightweight design make it ideal for all lev-
els of activity.

OPERATION

With every step, air is compressed from the
foot aircell to the Achilles aircell. Air flows
back to the foot aircell between steps. This
intermittent pulsating compression mas-
sages the fascia and tendon, providing
support and reducing swelling.

92 River Road
Summit, NJ/USA 07901
Tel: (800) 526-8785
(908) 273-6349
Fax: (800) 457-4221
(908) 273-1060

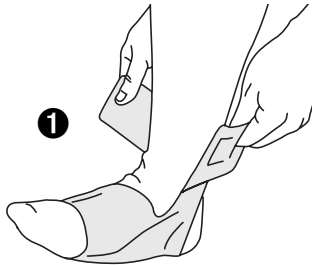
0049 8035 87 99 87
Fax: 00800-00 85 40 15 (free fax)
0049 8035 87 99 50
Tel: 00800-00 85 24 33 (free phone)
D-83115 Neubuurn
Georg-Wiesböck-Ring 12

Instructions

The AirHeel may be worn on either the left or the right foot.

Put on sock. Insert foot into AirHeel, toe
first. Position heel in opening in back of
wrap. Position Achilles aircell over tendon.
The height of the Achilles aircell may be
repositioned by detaching and firmly re-
attaching in place.

Place foot flat on floor and secure ankle
strap. Make sure that foot aircell is located
under arch of foot. To adjust AirHeel fit,
tighten/loosen ankle strap.

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CARE
The entire AirHeel can be hand washed in luke warm wa-
ter with mild soap and air dried.

HIGH ALTITUDE
At high altitudes the aircells will expand beyond their
optimal level. Adjust aircell pressure by tightening or loos-
ening strap.

LATEX
All Aircast products are latex-free.

WARRANTY POLICY
Aircast will provide a replacement to the original pur-
chaser should the AirHeel become unserviceable for any
reason for a period of 90 days from date of sale, provided
the defective part is returned to Aircast for analysis.